OMRON

# OMRON Comfort Cuff

Comfort Cuff

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
# OMRON

## Comfort Cuff

### Instruction Manual for OMRON's Comfort Cuff

Thank you for purchasing the OMRON Comfort Cuff. Arm circumference for this cuff is 22 to 42 centimeters. OMRON's Comfort cuff can be connected to various OMRON Blood Pressure Monitors. Please refer to the original instruction manual of your Blood Pressure Monitor.

#### Packaging Contents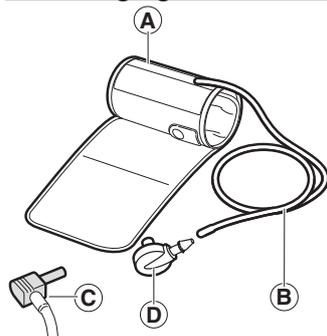

- A. Arm cuff (arm circumference 22-42 cm)
- B. Air tube
- C. Air plug
- D. Air plug


#### Instructions for use:

**Note: Connect correct air plug to the tube according to your device.**


#### Applying the Arm Cuff

You can wrap the cuff either on your right or left arm.

- Remove tight-fitting clothing from your upper arm.
- Do not place the cuff over thick clothes and do not roll up your sleeve if it is too tight.

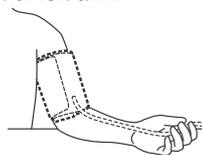
1. Insert the air plug into the air jack (on the left side of the device).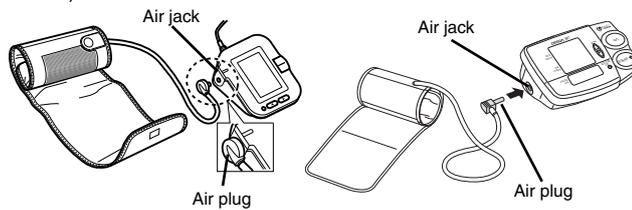


The cuff must be fully deflated when it is inserted into the air jack.


2. Sit in a chair with your feet flat on the floor and place your arm on a table so that the cuff will be at the same level as your heart.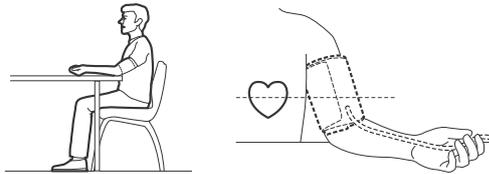

3. Hold the grip on the cuff securely with your hand.

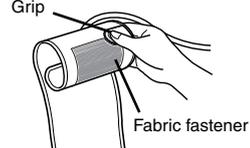
4. Turn the palm of your hand upward.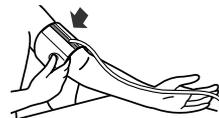

EN

5. Turn the palm of your hand upward.

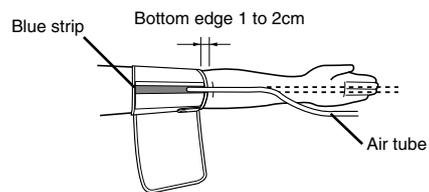Apply the cuff to your upper arm so that the blue strip is centred on the middle of your inner arm and points down the inside of the arm. The air tube should run down the inside of your forearm and be in line with your middle finger. The bottom of the cuff should be approximately 1 to 2 cm above the elbow.

6. When the cuff is positioned correctly, close the fabric fastener **FIRMLY**.

**Notes:**


- Make certain the cuff fits snugly around your arm.
- The cuff should make good contact with your skin. You should be able to fit your index finger between the cuff and your arm easily, so you can pull the cuff off and on.
- Make sure that there are no kinks in the air tubing.

### ***Taking measurements on the right arm***

Please note the following points when applying the cuff to your right arm. Apply the cuff so that the air tube is at the side of your elbow.

#### **Notes:**

- Be careful not to rest your arm on the air tube, or otherwise restrict the flow of air to the cuff.
- Apply the arm cuff so that no part of the cuff is positioned over the elbow joint. The cuff should be 1 to 2 cm above the elbow.
- If used 2 times per day, the cuff will last for 3 years.

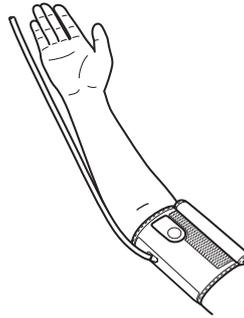### **Maintenance**

Stains on the cuff can be carefully removed using a moist cloth and soapy water.

#### **Do not wash the cuff or make it wet!**

Do not fold the cuff or air tube together too tightly and do not crease them.

EN

This cuff fulfils the provisions of EC directive 93/42/EEC (Medical Device Directive).

<b>Manufacturer</b> 	<b>OMRON HEALTHCARE Co., Ltd.</b> 53, Kunotsubo, Terado-cho, Muko, Kyoto, 617-0002 JAPAN		
<b>EU-representative</b> <table border="1"><tr><td>EC</td><td>REP</td></tr></table>	EC	REP	<b>OMRON HEALTHCARE EUROPE B.V.</b> Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS <a href="http://www.omron-healthcare.com">www.omron-healthcare.com</a>
EC	REP		
<b>Exclusive distributor in Russia &amp; importer</b>	<b>ComplectService Ltd.</b> 13-14 building, 26 B. Tishinskiy per., Moscow 123557, RUSSIA <a href="http://www.csmedica.ru">www.csmedica.ru</a>		
<b>Production facility</b>	<b>OMRON (DALIAN) CO., LTD.</b> Dalian, CHINA		

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